

USER GUIDE

3 Airflow Levels + Timer Control



3 Airflow Levels · 1H / 2H / 4H Timer · 1 or 2 Hoses · No Added Heat



WARNING: READ BEFORE USE

Keep the blower, timer controller and power adapter away from water. Do not immerse electrical components.

BEFORE YOU START

Important safety instructions and intended-use information

WARNING

- Keep the blower, timer controller and power adapter away from water.
- Do not immerse electrical components or use them with wet hands.
- Do not use damaged cords, plugs or connectors. Stop use if you notice unusual heat, odor, smoke or abnormal noise.

CAUTION

- Indoor use only. Keep the air inlet and outlet unobstructed.
- Disconnect power before filter replacement, cleaning or storage.
- Keep out of reach of children.
- Use only with compatible CPAP hoses and accessories.

NOTICE: INTENDED USE

- This product is designed to dry CPAP hoses using room-temperature airflow. It does not clean, sanitize, disinfect or sterilize CPAP equipment.
- It does not replace the cleaning, replacement or care schedule recommended by your CPAP equipment manufacturer.
- If you are unsure whether your hose or equipment is compatible with this dryer, follow the equipment manufacturer's instructions or contact your equipment provider.

CONTROLLER DEFAULTS



STARTUP DEFAULTS

First press powers on at Low (L). Timer default at startup: No Timer (continuous operation).

WHAT'S INCLUDED & CONNECTION

Check the parts, then connect in the order shown



Power Adapter



Timer Controller



Dryer Blower



Straight Adapter



Y Adapter



Replacement Filters

CONNECTION ORDER



Adapter Options

Use the Straight Adapter for one hose, or the Y Adapter for two hoses.



Y Adapter
(for 2 hoses)



Straight Adapter
(for 1 hose)

! HEATED HOSES

For compatible heated hoses, connect the round end without electronic contacts or a chip to the Straight or Y Adapter.

TIMER CONTROLLER

Two independent controls: airflow and timer



LOWER BUTTON · POWER / AIRFLOW

Low → Medium → High → Off

Indicators: Green = L

Yellow = M · Red = H



UPPER BUTTON · TIMER

No Timer → 1H → 2H → 4H → No Timer

Indicators: Green = 1H

Yellow = 2H · Red = 4H

HOW TO USE

- 1 POWER / AIRFLOW** First press: start at Low (L). Each additional press cycles Medium (M) → High (H) → Off.
- 2 TIMER** Select 1H, 2H or 4H. The blower stops automatically when the selected time ends. No Timer allows continuous operation.
- 3 RESET OR CANCEL** Selecting another 1H, 2H or 4H setting starts a new countdown. Selecting No Timer cancels the countdown and keeps the blower running.

INDEPENDENT CONTROLS

- ! Changing between L, M and H does not restart the active timer. Changing the timer does not change the airflow level.

HOW TO DRY 1 OR 2 HOSES

Choose the adapter that matches your setup

1 HOSE
Straight Adapter



2 HOSES
Y Adapter



Use the Straight Adapter for a single hose.

Both hoses share airflow, so drying two hoses may take longer.

QUICK DRYING STEPS

- 1 SHAKE OUT WATER** Remove as much excess water as possible before drying.
- 2 CONNECT** Use the Straight Adapter for one hose or the Y Adapter for two hoses. Keep each free hose end open and pointing downward.
- 3 SELECT AIRFLOW** Press POWER / AIRFLOW to start at Low (L), then press again to select Medium (M) or High (H).
- 4 SET TIMER** Select 1H, 2H or 4H. No Timer means continuous operation.
- 5 CHECK DRYNESS** Make sure each hose is completely dry before use. Run another cycle if needed.

DRYING TIME

One hose typically dries in about 30–45 minutes after excess water is shaken out. Time varies with hose type and length, remaining moisture, airflow, humidity and room conditions. Two hoses may take longer. A hose may dry before the selected timer cycle ends.


COMPATIBILITY & FILTER CARE

Simple maintenance for reliable airflow

HOSE COMPATIBILITY

STANDARD 22 MM	Designed for common 22 mm standard round hose ends.
MANY 15 MM SLIM	Fits many 15 mm slim hoses with compatible round cuffs. Fit depends on cuff shape and dimensions.
HEATED HOSES	Use the compatible round end without electronic contacts or a chip. Do not connect the electronic-contact end.

FILTER CARE

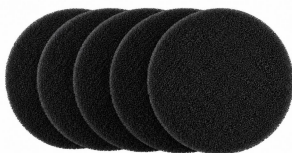

- Keep a filter installed during use.
- Check the filter regularly.
- Replace the filter if visibly dirty, clogged or damaged.
- Replace the filter pad every 3 months for routine care, or sooner if needed.

FILTER PAD INSTALLATION



DISCONNECT POWER

Turn off the dryer and unplug the power adapter before installing or replacing the filter pad.

- 1. Loosen the grille screws.** Do not remove them.
- 2. Position the filter pad.** Remove the old pad, if present, and place the new pad under the grille.
- 3. Tighten the screws.** Secure the grille in place.

TROUBLESHOOTING

Try these quick checks before contacting support

PROBLEM	WHAT TO CHECK
Dryer does not start	Confirm power connections. Press POWER / AIRFLOW once to start at L.
No timer indicator is lit	No Timer means continuous operation. Press TIMER to select 1H, 2H or 4H.
Dryer stopped automatically	The selected timer likely finished. Restart and choose a new setting if more drying is needed.
Airflow feels weak	Check the filter, hose connection, and air inlet/outlet for blockage. If using the Y Adapter, airflow is shared between both hoses.
Timer reset or cancelled	Selecting 1H, 2H or 4H starts a new countdown. Selecting No Timer cancels it; the blower keeps running.
Unusual noise, odor, or heat	Turn off and unplug the dryer. Stop use and contact MOZZPAD Support.

! PROBLEM NOT LISTED OR STILL NOT RESOLVED?
support@mozzpad.com • www.mozzpad.com

STOP USE IF NECESSARY
! If you notice smoke, unusual odor, excessive heat, or abnormal noise, unplug the product and contact MOZZPAD Support.

SUPPORT & PARTS REPLACEMENT

Help with setup, replacement parts and product questions

CUSTOMER SUPPORT

Email: support@mozzpad.com

Website: www.mozzpad.com



SCAN FOR DIGITAL SUPPORT

[Guides](#) · [Videos](#) · [Troubleshooting](#)

1-YEAR REPLACEMENT-PART SUPPORT

MOZZPAD provides one-year replacement-part support for eligible included components: the blower, power adapter and timer controller.

Contact MOZZPAD Support to check eligibility and the support terms applicable to your purchase.

REQUESTING SUPPORT

1. Have your order number ready.
2. Describe the issue. Include clear photos or a short video, if available.
3. Email support@mozzpad.com. Include the blower model and the power adapter output printed on your labels.

REPLACEMENT PARTS AFTER YEAR ONE

Replacement parts, including the blower, power adapter and timer controller, remain available for purchase after the one-year support period. Contact MOZZPAD Support for compatible parts, availability and pricing.

! IMPORTANT USE NOTICE

This product dries CPAP hoses using airflow only. It does not sanitize, disinfect or sterilize CPAP equipment. Follow your CPAP equipment manufacturer's instructions for cleaning, replacement and care.

PRODUCT MARKINGS & ADAPTERS

Match the blower model to its supplied power adapter



CXB7530B12H

Power adapter: 12V / 1A

CXB9733B12VH

Power adapter: 12V / 2A

CXB9733B12UH

Power adapter: 12V / 3A

Blower model and power adapter vary by product configuration. Use only the power adapter supplied with your product.

© 2026 MOZZPAD. All rights reserved.