

# USER GUIDE

## 3 Airflow Levels + Timer Control



**3 Airflow Levels · 1H / 2H / 4H Timer · 1 or 2 Hoses · No Added Heat**

### **! WARNING: READ BEFORE USE**

Keep the blower, timer controller and power adapter away from water. Do not immerse electrical components.

---

# BEFORE YOU START

---

Important safety instructions and intended-use information

## WARNING

- Keep the blower, timer controller and power adapter away from water.
- Do not immerse electrical components or use them with wet hands.
- Do not use damaged cords, plugs or connectors. Stop use if you notice unusual heat, odor, smoke or abnormal noise.

## CAUTION

- Indoor use only. Keep the air inlet and outlet unobstructed.
- Disconnect power before filter replacement, cleaning or storage.
- Keep out of reach of children.
- Use only with compatible CPAP hoses and accessories.

## NOTICE: INTENDED USE

- This product is designed to dry CPAP hoses using room-temperature airflow. It does not clean, sanitize, disinfect or sterilize CPAP equipment.
- It does not replace the cleaning, replacement or care schedule recommended by your CPAP equipment manufacturer.
- If you are unsure whether your hose or equipment is compatible with this dryer, follow the equipment manufacturer's instructions or contact your equipment provider.

## CONTROLLER DEFAULTS



### STARTUP DEFAULTS

Press POWER / AIRFLOW once to start at Low (L). Timer default at startup: No Timer (continuous operation).

# WHAT'S INCLUDED & CONNECTIONS

Check the parts, then connect in the order shown



**Power Adapter**



**Timer Controller**



**Dryer Blower**



**Straight Adapter**



**Y Adapter**



**Replacement Filters**

## CONNECTION ORDER



### Adapter Options

Use the Straight Adapter for one hose, or the Y Adapter for two hoses.



**Y Adapter**  
(for 2 hoses)



**Straight Adapter**  
(for 1 hose)



### HEATED HOSES

For compatible heated hoses, connect the round end without electronic contacts or a chip to the Straight or Y Adapter.

# TIMER CONTROLLER

Two independent controls: airflow and timer



## LOWER BUTTON · POWER / AIRFLOW

Low → Medium → High → Off

Indicators: Green = L

Yellow = M · Red = H



## UPPER BUTTON · TIMER

No Timer → 1H → 2H → 4H → No Timer

Indicators: Green = 1H

Yellow = 2H · Red = 4H

## HOW TO USE

- 1 POWER / AIRFLOW** First press: start at Low (L). Each additional press cycles through Medium (M) → High (H) → Off.
- 2 TIMER** Select 1H, 2H or 4H. The blower stops automatically when the selected time ends. No Timer allows continuous operation.
- 3 RESET OR CANCEL** Selecting another 1H, 2H or 4H setting starts a new countdown. Selecting No Timer cancels the countdown and keeps the blower running.

### INDEPENDENT CONTROLS

- ! Changing between L, M and H does not restart the active timer. Changing the timer does not change the airflow level.

# HOW TO DRY 1 OR 2 HOSES

Choose the adapter that matches your setup

**1 HOSE**  
Straight Adapter



**2 HOSES**  
Y Adapter



Use the Straight Adapter for a single hose.

Both hoses share airflow, so drying two hoses may take longer.

## QUICK DRYING STEPS

- 1 SHAKE OUT WATER** Remove as much excess water as possible before drying.
- 2 CONNECT** Use the Straight Adapter for one hose or the Y Adapter for two hoses. Keep each free hose end open and pointing downward.
- 3 SELECT AIRFLOW** Press POWER / AIRFLOW to start at Low (L), then press again to select Medium (M) or High (H).
- 4 SET TIMER** Select 1H, 2H or 4H. No Timer means continuous operation.
- 5 CHECK DRYNESS** Make sure each hose is completely dry before use. Run another cycle if needed.

### DRYING TIME

Drying time varies with hose type and length, remaining moisture, airflow setting, humidity and room conditions. Shake out as much excess water as possible before drying. Two hoses may take longer.

# COMPATIBILITY & FILTER CARE

Simple maintenance for reliable airflow

## HOSE COMPATIBILITY

|                        |                                                                                                                |
|------------------------|----------------------------------------------------------------------------------------------------------------|
| <b>STANDARD 22 MM</b>  | Designed for common 22 mm standard round hose ends.                                                            |
| <b>MANY 15 MM SLIM</b> | Fits many 15 mm slim hoses with compatible round cuffs. Fit depends on cuff shape and dimensions.              |
| <b>HEATED HOSES</b>    | Use the compatible round end without electronic contacts or a chip. Do not connect the electronic-contact end. |

## FILTER CARE



- Keep a filter installed during use.
- Check the filter regularly.
- Replace the filter if visibly dirty, clogged or damaged.
- Replace the filter pad every 3 months for routine care, or sooner if needed.

## FILTER PAD INSTALLATION

### ! DISCONNECT POWER

Turn off the dryer and unplug the power adapter before installing or replacing the filter pad.

- 1. Loosen the grille screws.** Do not remove them.
- 2. Position the filter pad.** Remove the old pad, if present, and place the new pad under the grille.
- 3. Tighten the screws.** Secure the grille in place.

# TROUBLESHOOTING

Try these quick checks before contacting support

| PROBLEM                                       | WHAT TO CHECK                                                                                                                       |
|-----------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------|
| Dryer does not start                          | Confirm power connections. Press POWER / AIRFLOW once to start at L.                                                                |
| No timer indicator is lit                     | No Timer means continuous operation. Press TIMER to select 1H, 2H or 4H.                                                            |
| Dryer stopped automatically                   | The selected timer likely finished. Restart and choose a new setting if more drying is needed.                                      |
| Airflow feels weak                            | Check the filter, hose connection, and air inlet/outlet for blockage. If using the Y Adapter, airflow is shared between both hoses. |
| Timer reset or cancelled                      | Selecting 1H, 2H or 4H starts a new countdown. Selecting No Timer cancels it; the blower keeps running.                             |
| Unusual noise, odor, smoke, or excessive heat | Turn off and unplug the dryer. Stop use and contact MOZZPAD Support.                                                                |

**! PROBLEM NOT LISTED OR STILL NOT RESOLVED?**  
support@mozzpad.com • www.mozzpad.com

**STOP USE IF NECESSARY**  
**!** If you notice smoke, unusual odor, excessive heat, or abnormal noise, unplug the product and contact MOZZPAD Support.

# SUPPORT & PARTS REPLACEMENT

Help with setup, replacement parts and product questions

## CUSTOMER SUPPORT

Email: [support@mozzpad.com](mailto:support@mozzpad.com)

Website: [www.mozzpad.com](http://www.mozzpad.com)



## SCAN FOR DIGITAL SUPPORT

[Guides](#) · [Videos](#) · [Troubleshooting](#)

## 1-YEAR REPLACEMENT-PART SUPPORT

MOZZPAD provides one-year replacement-part support for eligible included components: the blower, power adapter and timer controller.

Contact MOZZPAD Support to check eligibility and the support terms applicable to your purchase.

## REQUESTING SUPPORT

1. Have your order number ready.
2. Describe the issue. Include clear photos or a short video, if available.
3. Email [support@mozzpad.com](mailto:support@mozzpad.com). Include the blower model and the power adapter output printed on your labels.

## REPLACEMENT PARTS AFTER YEAR ONE

After the one-year support period, compatible replacement parts may be available for purchase. Contact MOZZPAD Support for availability and pricing.



### IMPORTANT USE NOTICE

This product dries CPAP hoses using airflow only. It does not sanitize, disinfect or sterilize CPAP equipment. Follow your CPAP equipment manufacturer's instructions for cleaning, replacement and care.



# BLOWER MODELS & POWER ADAPTERS

Match the blower model to its supplied power adapter



**CXB7530B12H**

Power adapter: 12V / 1A

**CXB9733B12VH**

Power adapter: 12V / 2A

**CXB9733B12UH**

Power adapter: 12V / 3A

Blower model and power adapter vary by product configuration. Use only the power adapter supplied with your product or a MOZZPAD-approved compatible replacement matched to your blower model.

© 2026 MOZZPAD. All rights reserved.